

Cleaning Instructions For The Backbone™ Positioning Cushion



CLEANING STANDARD

The BackBone™ Positioning Wedge must be wiped clean with institutional wipes approved disinfectant wipes daily and when visibly soiled.

May rinse with soap & water if heavily soiled.

- a. Use institutional approved wipes (bleach, hydrogen peroxide, Alcohol) between uses and whenever soiled. (Incontinence wipes; “baby wipes” are not effective to provide disinfection)

Ensure the wedge is wiped wet enough to accomplish **1 minute dwell time** required to achieve sanitation.

- b. Allow to Air Dry before re-use

ISOLATION ROOM CLEANING

- a. Use Clorox Bleach cleaning wipes between uses and whenever soiled.

Ensure the wedge is wiped wet enough to accomplish **3 minute dwell time** required to achieve sanitation.

Discard Cushion if contamination is suspected

- ✓ Discard if cover is damaged (torn, punctured, opened seams, etc)
- ✓ Discard if submerged under any type of fluid or stool
- ✓ Discard if contamination is suspected for any reason